


Instructions for the Collection of Convicted Offender DNA Samples

1. Swab inside of cheek using **ALL Four** buccal swabs provided in kit.




2. Insert **UNWRAPPED** swabs into **swab envelope**, **swab head first**.

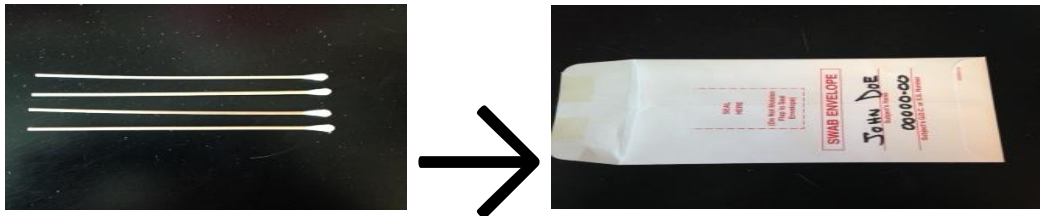

3. Seal swab envelope with **RED Evidence tape only**, **DO NOT MOISTEN ADHESIVE** to seal.



4. Place DOC submission form and swab envelope into shipping envelope. Enter return address.

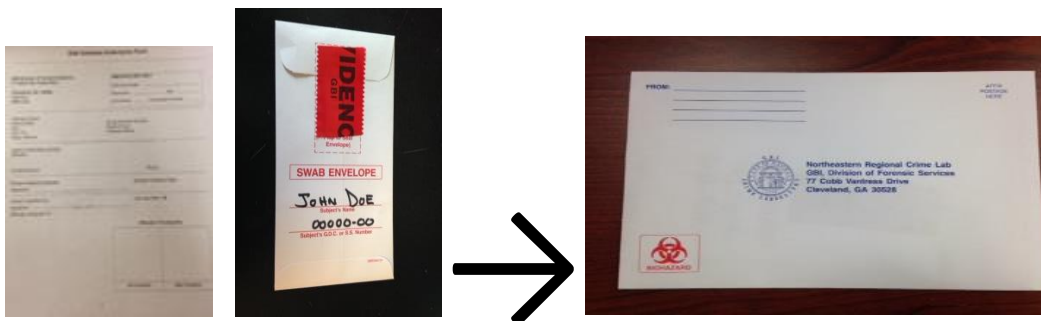

5. Seal shipping envelope with Kit Sealing Tape and mail to GBI.

